

# SEAL CHECKIT

TEST SYSTEM FOR DAILY TESTING OF SEALING DEVICES AND  
FOR OPERATIONAL QUALIFICATION ACCORDING TO EN ISO 11607-2.

**GUIDELINES OF THE WFHSS** ADVISE TO TEST EACH MEDICAL HEAT SEALER AT THE BEGINNING OF THE DAY, AFTER THE HEATING UP.

BASED UPON THE TEST THE HEAT SEALER CAN BE RELEASED FOR THE DAY.

GANDUS SALDATRICI SEAL CHECKIT IS THE PERFECT WAY TO COMPLY TO THE ISO 11607.2 REQUIREMENTS FOR OPERATIONAL QUALIFICATION (OQ) AND DAILY TESTING OF YOUR MEDICAL HEAT SEALERS

- IS THE SEALING TEMPERATURE DEFINED AND CORRECT?
- ARE SPEED AND FORCE THE MOST SUITABLE FOR THAT KIND OF PRODUCT?
- DID THE MANUFACTURER INDICATE PARAMETERS INSIDE THE SPECS?

## HOW DOES IT WORKS?

PLACE IN BETWEEN THE LAYERS OF MEDICAL POUCHES WITH FACE UP TO THE LAMINATE SIDE OF THE POUCH. THE COMBINATION IS THEN HEAT-SEALED.

BY ITS UNIQUE PRINTED PATTERN OF THE BLACK AREA THE INSPECTION OF THE SEAL AND THE DETECTION OF MINOR OR BIGGER PROBLEMS ARE EASILY DETECTED.

- SEALING TEMPERATURE TOO LOW
- SEALING PRESSURE TOO LOW
- INTERRUPTIONS IN THE SEAL

## COMPLIANCE TO STANDARDS

GANDUS SALDATRICI SEAL CHECKIT IS COMPLIANT TO THE FOLLOWING STANDARD:

PROCEDURE: ISO 11607 - 2

MATERIALS: ISO 868 - 3

## TECHNICAL DETAILS EACH TEST:

DIMENSIONS: 175 x 75 MM

PAPER: 60 GR/M<sup>2</sup>, MEDICAL GRADE

INKS: WATERBASED, NON SOLVENT, NON-TOXIC, NON-HEAVY METALS

